



# Instructions for Use **Note:** Review cautions and warnings related to the Knee Separator device before use.

## Intended Use

The Knee Separator is a foam cushion covered with soft material intended to be placed between the knee(s) of a patient for prevention of pressure injuries/ulcers. It may be used for one or more of the following:

- To reduce pressure on or between the knees
- To aid in the prevention of pressure injuries/pressure ulcers

The Knee Separator is used to eliminate pressure along with shear and friction forces between the knees and relieve undue discomfort.

## Application of the Knee Separator:

1. Place the Knee Separator between the patient's knees in the appropriate manner when lying in bed.
2. Remove the Knee Separator from between the knees before getting out of bed.



## Indications for Use of the Knee Separator

- Indications
  - Prevention of pressure injuries.
  - Cushioning and offloading pressure between the knees
- Contraindications
  - Fractures
  - Excessive swelling
  - Infection
  - Open wound
  - Skin rash
  - Atrophic skin
  - Surgical sites

## Cautions

- The Knee Separator should be removed every 8 hours to inspect skin.
- In case of improper handling, the function and maximum safety of the product can no longer be guaranteed.
- This aid is intended for single patient - multiple use only.

## Warnings

- Make sure the Knee Separator is appropriately placed between the knees.
- Use under a physician's supervision.
- Avoid use of the Knee Separator if allergic to the material.
- Do not use over open wound or injured skin.
- Discontinue the Knee Separator with any associated irritation/pressure/discomfort/pain.

## Technical Data

### Size

- One size fits most.

### Color

- Beige

**GMDN Code:** \*41542-Leg/Knee Cushion

### Material

- Latex-free polyurethane foam
- Polyester fabric and fleece cover material
- Metal-free

### Size

9.5in (24cm) (L)x 6in (15cm) (H) x 3in (7.5cm) (H-Center) x 8.2in (21cm) (W)

### Construction Features

- Knee Separator is for cushioning and offloading the knees.

### Maintenance

- The product does not require maintenance aside from disinfecting or washing and drying as indicated.
- Single Patient - Multiple Use.

## Cleaning Instructions

- To hand wash: Use mild, non-bleach cleanser and warm water.
- To machine wash and dry: wash in warm water (max. temperature 60°C/140°F) on gentle cycle and dry on low heat or air dry.
- To sanitize: Use an antiseptic wipes or spray.
- Dry completely before re-using.



## Single Patient - Multiple Use

ES	Paciente único - Uso múltiple
DE	Einzelpatient – Mehrfachverwendun
DA	Enkelt patient - flergangsbrug
FR	Patient unique - Usage multiple
IT	Singolo paziente - Uso multiplo
LT	Vienas pacientas – daugkartinis naudojimas
NL	Eén patiënt - meervoudig gebruik
NO	Enkeltpasient - Flerbruk
PL	Jeden pacjent — wiele zastosowań
PT	Paciente Único - Uso Múltiplo
PT-B	Paciente único - Uso múltiplo
RO	Pacient unic - Utilizare multiplă
SV	Enstaka patient - Flera användningsområden
SK	Jeden pacient – viacnásobné použitie
SL	En bolnik - večkratna uporaba
TL	bawat pasyente - ng iba
TR	Tek Hasta - Çoklu Kullanım



## Consult Instructions for Use

ES	Consultar instrucciones de uso
DE	Konsultieren Sie die Gebrauchsanweisung lesen
DA	Se brugsanvisningen
FR	Consulter les instructions d'utilisation
IT	Consultare le istruzioni per l'uso
LT	Pasikonsultuokite naudoti naudojimą instrukcijas
NL	Raadpleeg de instructies voor gebruik
NO	Kontakt instruksjonene for bruk
PL	Skonsultuj się z instrukcjami do użycia
PT	Consulte as instruções de uso
PT-B	Consulte as instruções de uso
RO	Consultați instrucțiunile de utilizare
SV	Konsultera bruksanvisningen
SK	Konzultujte pokyny na použitie
SL	Oglejte si navodila za uporabo
TL	Sasangguni tagubilin gamitin
TR	Kullanım için talimatlara danışın



## Unique Device Identifier

ES	Identificador de dispositivo único
DE	Eindeutige Geräteerkennung
DA	Unikt enheds Identifikator
FR	Identifiant unique de l'appareil
IT	Identificatore univoco del dispositivo
LT	Unikalus įrenginio identifikatorius
NL	Unieke apparaatindicator
NO	Unik enhetsidentifikator
PL	Unikalny identyfikator urządzenia
PT	Identificador Único de Dispositivo
PT-B	Indicador único de dispositivo
RO	Identificator unic de dispozitiv
SV	Unik enhetsidentifierare
SK	Jedinečný identifikátor zariadenia
SL	Edinstven indikator naprave
TL	Tulali banta identifier
TR	Benzersiz Cihaz Tanımlayıcı



## Latex-Free (Not Made with Natural Rubber Latex)

ES	Sin látex
DE	Latexfrei
DA	Latex fri
FR	Sans latex
IT	Senza latex
LT	Be latekso
NL	Latex vrij
NO	Uten latex
PL	Bez lateksu
PT	Látex grátis
PT-B	Sem látex
RO	Fără latex
SV	Latex fri
SK	Bez latexu
SL	Brez lateksa
TL	Walang Latex
TR	Lateks içermez



## Medical Device

ES	Dispositivo médico
DE	Medizinprodukt
DA	Medicinsk udstyr
FR	Dispositif médical
IT	Dispositivo medico
LT	Medicininis prietaisas
NL	Medisch apparaat
NO	Medisinsk enhet
PL	Urządzenie medyczne
PT	Aparelho médico
PT-B	Aparelho médico
RO	Aparat medical
SV	Medicinsk utrustning
SK	Lekárska pomôcka
SL	Medicinski pripomoček
TL	Gamit Medikal
TR	Tıbbi cihaz

## Leg/Knee Cushion\*

ES	Cojín para piernas/rodillas
DE	Beinkissen
DA	Benstøttepude
FR	Coussin pour genou/jambe
IT	Cuscino per arti inferiori
LT	Spilvens kājām
NL	Beenkussen
NO	Pute, støtte, bein
PL	Poducha pod nogę
PT	Almofada para as pernas
PT-B	Almofada para as pernas
RO	Pernuță pentru picior
SV	benkudde
SK	Poduška pre nohy; vankúš na koleno nohy
SL	Blazina za noge
TL	
TR	bacak/diz yastığı



## Keep Dry and Away from Direct Sunlight

ES	Manténgase seco y alejado de la luz solar directa
DE	Trocken und vor direkter Sonneneinstrahlung schützen
DA	Hold dig tør og væk fra direkte sollys
FR	Garder sèche et loin de la lumière directe du soleil
IT	Mantieniti asciutto e lontano dalla luce solare diretta
LT	Laikykite sausas ir toli nuo tiesioginių saulės spindulių
NL	Droog en uit de buurt van direct zonlicht houden
NO	Hold deg tørr og vekk fra direkte sollys
PL	Przechowywać w suchym miejscu i z dala od bezpośredniego światła słonecznego
PT	Mantenha seco e longe da luz solar direta
PT-B	Manter seco e afastado da luz direta do sol
RO	Păstrați uscare și departe de lumina directă a soarelui
SV	Håll torr och bort från direkt solljus
SK	Uchovávať v suchu a mimo dosahu priameho slnečného svetla
SL	Hranite suho in stran od neposredne sončne svetlobe
TL	At ang tuyong ang magtuturo sa sunlight
TR	Kuru ve doğrudan güneş ışığından uzak durun

## Knee Separator

ES	separador de rodilla
DE	Knieschleifer
DA	knæ separator
FR	séparateur de genoux
IT	separatori di ginocchia
LT	kelio separatorius
NL	kniescheider
NO	kneseparator
PL	separator kolan
PT	separador de joelho
PT-B	separador de joelho
RO	separator de genunchi
SV	knäavskiljare
SK	kolenný separátor
SL	separator za kolena
TL	
TR	diz ayırıcı