



ULNAR NERVE PROTECTOR



ULNAR NERVE PROTECTOR

GMDN Code: 44284

ARM FRICTION/IMPACT
PROTECTOR*

Consult Instructions for Use 

 Walgreen Health Solutions, LLC
1316 Sherman Avenue
Evanston, IL 60201
Phone: (847) 328-9540
Email: info@walgreenhealthsolutions.com
www.walgreenhealthsolutions.com Made in China

Intended Use:

The Ulnar Nerve Protector is a device designed to be worn over the elbow to provide a cushion to protect the ulnar nerve from friction against surfaces and impacts against objects. Is a device designed to be worn over some length of the arm (various ranges from the base of the fingers to above the elbow) to provide a cover and/or cushion to protect the limb and its skin from friction and knocks against objects. It is typically made from soft padded materials and held in place with a strap. It is designed to cover only the elbow joint to protect the ulnar nerve and is not intended to cover a larger portion of the arm. The Ulnar Nerve Protector provides cushioning on the elbow to protect the ulnar nerve. It is typically used by persons who are at-risk for pressure injuries/ ulcers on their elbows, bedridden, or prone to lack of control with their arms causing injury. This is a single-patient/multiple-use product.



1 Instructions for Use **Note:** Review cautions and warnings related to the Ulnar Nerve Protector device before selection and use.

The Ulnar Nerve Protector to offload the ulnar nerve and protect the nerve from injury.

Application of the Ulnar Nerve Protector:

1. Open the Ulnar Nerve Protector so that the hook-and-loop strap is pulled away from the foam pad.
2. Position the bony prominence of the elbow appropriately into the center of the pad.
3. Secure the hook-and-loop strap over the fold of the elbow.
4. Use the two-finger test to ensure proper fit of the Ulnar Nerve Protector. (Make sure the strap is not too tight and not too loose by using the Two-Finger Test.)

2 Indications for Use of the Ulnar Nerve Protector Pad

- Indications
 - Prevention of pressure injuries/ulcers.
 - Cushioning device to protect the ulnar nerve.
- Contraindications
 - Fractures
 - Excessive swelling
 - Infection
 - Skin rash
 - Atrophic skin
 - Surgical sites
 - Open Wound

3 Cautions

- The Ulnar Nerve Protector Pad should be removed every 8 hours to inspect skin.
- In case of improper handling, the function and maximum safety of the product can no longer be guaranteed.
- This aid is intended for single patient - multiple use only.

4 Warnings

- Make sure the Ulnar Nerve Protector pad is appropriately applied.
- Make sure strap is not too loose or too tight to prevent injury from pressure, shear and friction forces.
- Use under a physician's supervision. Contact physician before applying.
- Avoid use of the Ulnar Nerve Protector Pad if allergic to the material.
- Do not use over open wound or injured skin.
- Discontinue the Ulnar Nerve Protector with any associated irritation/pressure/discomfort/pain.

5 Technical Data

Size

- One size fits most.

Color

- Beige

GMDN Code: 44284*

** CA Prop 65 Compliant

Material

- Protector Pad: Latex-free Polyurethane Foam
- Hook-and-Loop Fastener: Nylon + Flame Retardant (TCPP)**

Size

- 12.2in (31cm) L x 8.3in (21cm) W x 2.2in (5cm) W
- Thickness 0.98in (2.5cm)

Maintenance

- The product does not require maintenance aside from disinfecting or washing and drying as indicated.
- Single patient - Multiple Use.

Construction Features

- Ulnar Nerve protector pad for offloading the ulnar nerve.
- Fixation of the pad with hood-and-loop strap.
- Not made with natural rubber latex.

6 Cleaning Instructions

- To hand wash: Use mild, non-bleach cleanser and warm water.
- To machine wash and dry: Secure straps and place in a net laundry bag or pillowcase and wash in warm water (60°C/140°F) on gentle cycle and dry on low heat.
- To sanitize: Use an antiseptic wipes or spray.
- Dry completely before re-applying.



Single Patient - Multiple Use

ES	Paciente único - Uso múltiple
DE	Einzelpatient – Mehrfachverwendun
DA	Enkelt patient - flergangsbrug
FR	Patient unique - Usage multiple
IT	Singolo paziente - Uso multiplo
LT	Vienas pacientas – daugkartinis naudojimas
NL	Eén patiënt - meervoudig gebruik
NO	Enkeltpasient - Flerbruk
PL	Jeden pacjent — wiele zastosowań
PT	Paciente Único - Uso Múltiplo
PT-B	Paciente único - Uso múltiplo
RO	Pacient unic - Utilizare multiplă
SV	Enstaka patient - Flera användningsområden
SK	Jeden pacient – viacnásobné použitie
SL	En bolnik - večkratna uporaba
TL	bawat pasyente - ng iba
TR	Tek Hasta - Çoklu Kullanım



Consult Instructions for Use

ES	Consultar instrucciones de uso
DE	Konsultieren Sie die Gebrauchsanweisung lesen
DA	Se brugsanvisningen
FR	Consulter les instructions d'utilisation
IT	Consultare le istruzioni per l'uso
LT	Pasikonsultuokite naudoti naudojimą instrukcijas
NL	Raadpleeg de instructies voor gebruik
NO	Kontakt instruksjonene for bruk
PL	Skonsultuj się z instrukcjami do użycia
PT	Consulte as instruções de uso
PT-B	Consulte as instruções de uso
RO	Consultați instrucțiunile de utilizare
SV	Konsultera bruksanvisningen
SK	Konzultujte pokyny na použitie
SL	Oglejte si navodila za uporabo
TL	Sasangguni tagubilin gamitin
TR	Kullanım için talimatlara danışın



Unique Device Identifier

ES	Identificador de dispositivo único
DE	Eindeutige Geräteerkennung
DA	Unikt enheds Identifikator
FR	Identifiant unique de l'appareil
IT	Identificatore univoco del dispositivo
LT	Unikalus įrenginio identifikatorius
NL	Unieke apparaatindicator
NO	Unik enhetsidentifikator
PL	Unikalny identyfikator urządzenia
PT	Identificador Único de Dispositivo
PT-B	Indicador único de dispositivo
RO	Identificator unic de dispozitiv
SV	Unik enhetsidentifierare
SK	Jedinečný identifikátor zariadenia
SL	Edinstven indikator naprave
TL	Tulali banta identifier
TR	Benzersiz Cihaz Tanımlayıcı



Latex-Free (Not Made with Natural Rubber Latex)

ES	Sin látex
DE	Latexfrei
DA	Latex fri
FR	Sans latex
IT	Senza latex
LT	Be latekso
NL	Latex vrij
NO	Uten latex
PL	Bez lateksu
PT	Látex grátis
PT-B	Sem látex
RO	Fără latex
SV	Latex fri
SK	Bez latexu
SL	Brez lateksa
TL	Walang Latex
TR	Lateks içermez

MD Medical Device

ES	Dispositivo médico
DE	Medizinprodukt
DA	Medicinsk udstyr
FR	Dispositif médical
IT	Dispositivo medico
LT	Medicininis prietaisai
NL	Medisch apparaat
NO	Medisinsk enhet
PL	Urządzenie medyczne
PT	Aparelho médico
PT-B	Aparelho médico
RO	Aparat medical
SV	Medicinsk utrustning
SK	Lekárska pomôcka
SL	Medicinski pripomoček
TL	Gamit Medikal
TR	Tıbbi cihaz

Elbow/Ankle Protector*

ES	Protector para codo
DE	Ellbogen-/Knöchelschutz
DA	Albuebeskyttelsesudstyr
FR	Protège-coude
IT	Protezione per gomito
LT	Alkūnių apsauga
NL	Elleboogbeschermer
NO	Beskytter, albue
PL	Ochraniacz stawu łokciowego
PT	Protector para cotovelo
PT-B	Protector para cotovelo
RO	Dispozitiv de protecție pentru cot
SV	armbågsskydd
SK	Chránič lakťa
SL	Ščitnik za komolec
TL	
TR	dirsek/ayak bileği koruyucusu

 **Keep Dry and Away from Direct Sunlight**

ES	Manténgase seco y alejado de la luz solar directa
DE	Trocken und vor direkter Sonneneinstrahlung schützen
DA	Hold dig tør og væk fra direkte sollys
FR	Garder sèche et loin de la lumière directe du soleil
IT	Mantieniti asciutto e lontano dalla luce solare diretta
LT	Laikykite sausas ir toli nuo tiesioginių saulės spindulių
NL	Droog en uit de buurt van direct zonlicht houden
NO	Hold deg tørr og vekk fra direkte sollys
PL	Przechowywać w suchym miejscu i z dala od bezpośredniego światła słonecznego
PT	Mantenha seco e longe da luz solar direta
PT-B	Manter seco e afastado da luz direta do sol
RO	Păstrați uscare și departe de lumina directă a soarelui
SV	Håll torr och bort från direkt solljus
SK	Uchovávať v suchu a mimo dosahu priameho slnečného svetla
SL	Hranite suho in stran od neposredne sončne svetlobe
TL	At ang tuyong ang magtuturo sa sunlight
TR	Kuru ve doğrudan güneş ışığından uzak durun

Ulnar Nerve Protector

ES	Protector del nervio cubital
DE	Ulnar-Nervenschutz
DA	Ulnar nervebeskytter
FR	Protecteur du nerf cubital
IT	Protettore del nervo ulnare
LT	Alkūninio nervo apsauga
NL	Ulnar zenuw beschermer
NO	Ulnar nervebeskytter
PL	Ochraniacz nerwu łokciowego
PT	Protector do nervo ulnar
PT-B	Protetor do nervo ulnar
RO	Protectorul nervului ulnar
SV	Ulnar nervskydd
SK	Chránič lakťového nervu
SL	Zaščita za ulnarni živec
TL	
TR	Ulnar Sinir Koruyucu