ELBOWLIFT[®]
S U S P E N S I O N P A D

ELBOWLIFT[®] SUSPENSION PAD
ELBOW PROTECTOR

1

Instructions for Use **(Note: Review cautions and warnings related to ELBOWLIFT® device before selection and use.**

The ELBOWLIFT® Suspension Pad protects the elbow from pressure ulcers.

Application of the ELBOWLIFT® Suspension Pad:

1. Open up the ELBOWLIFT® Suspension Pad so that the hook-and-loop strap is pulled away from the suspension pad.
2. Place tip of patient's elbow over the centering hole with the foam insert pads aligned under the arm and forearm.
3. Close the strap across the elbow using the hook-and-loop closure.
4. Fit the ELBOWLIFT® so it is not too tight but not too loose by using the Two-Finger Test.

2

Indications for Use of ELBOWLIFT® Suspension Pad

- Prevention of pressure injuries.
- Offloading pressure from the elbow.
- Offload pre-existing wound on the elbow.
- Contraindications
 - Fractures
 - Excessive swelling
 - Infection

3

Cautions

- The ELBOWLIFT® Suspension Pad should be removed every 6 hours to inspect skin or wound status. In case of improper handling, the function and maximum safety of the product can no longer be guaranteed. This aid is intended for single-patient use only.

4

Warnings

- Make sure the ELBOWLIFT® is appropriately applied so elbow is centered over the hole.
- Make sure straps are not too loose or too tight to prevent pressure, shear and friction forces.
- Use under a physician's supervision. Contact physician before applying to the elbow.
- Continue local wound care as prescribed by the attending physician.
- Avoid use of ELBOWLIFT® if allergic to the material.

5

Technical Data

Size	Material	Maintenance	Construction Features
<ul style="list-style-type: none"> • One size fits most. 	<ul style="list-style-type: none"> • Suspension Pad: Latex-free polyurethane foam • Hook-and-loop fastener: Nylon • Fabric: Polyester 	<ul style="list-style-type: none"> • The product does not require maintenance aside from disinfecting or washing and drying as indicated. • Single patient - Multiple Use 	<ul style="list-style-type: none"> • Elbow suspension pad for off-loading the area around the elbow. Fixation of the pad with dorsally thick foam covered strap.
Color <ul style="list-style-type: none"> • Beige 			

6

Cleaning Instructions

- To hand wash: Use mild, non-bleach cleanser and warm water.
- To machine wash and dry: Secure straps and place in net laundry bag or pillowcase and wash in warm water (40 - 60°C/105 -140°F) on gentle cycle and dry on low heat.
- To sanitize: Use an antiseptic wipes or spray.
- Dry completely before re-applying.

**Consult Instructions for Use
Available in Multiple languages at:**

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Walgreen Health Solutions, LLC
1316 Sherman Avenue
Evanston, IL 60201
Phone: (847) 328-9540
Email: info@walgreenhealthsolutions.com
www.walgreenhealthsolutions.com
Made in China

CH	REP
----	-----

MedEnvoy
Prinses Margrietplantsoen 33 - Suite 123
2595 AM The Hague
The Netherlands

Elbow Protector: GMDN Code 32858

ES	Protector de codo
DE	Ellbogenschutz
DA	Albuebeskytter
FR	Protecteur de coude
IT	Protezione del gomito
LT	Alkūnės apsauga
NL	Elleboogbeschermer
NO	Albuebeskytter
PL	Ochrona łokcia
PT	Protector de cotovelo
PT-B	Protetor de cotovelo
RO	Protecție pentru cot
SV	Armbågsskydd
SK	Ochrana lakťov
SL	Zaščita komolcev
TL	Siko tanggol
TR	Dirsek Koruyucu

EC	REP
----	-----

Emergo Europe B.V
Prinsessegracht 20
2514 AP The Hague
The Netherlands
**New Address Effective
2023-07-31:**
Emergo Europe
Westervoortsedijk 60
6827 AT Arnhem
The Netherland

Suspension Pad

ES	Almohadilla de suspensión
DE	Federungskissen
DA	Ophængningspude
FR	Coussinet de suspension
IT	Cuscinetto per sospensione
LT	Pakabos pagalvėlė
NL	Ophangkussen
NO	Ophengspute
PL	Poduszka do zawieszania
PT	Almofada de suspensão
PT-B	Almofada de suspensão
RO	Tampon de suspensie
SV	Upphängningsdyna
SK	Závesná podložka
SL	Blazinica za vzmetenje
TL	Sulatan Pagkatigil
TR	Süspansiyon pedi

Single Patient - Multiple Use

ES	Un solo paciente - Uso múltiple
DE	EinzelnerPatient – Mehrfachanwendbar
DA	Enkelt patient - flergangsbrug
FR	Un seul patient – à usage multiple
IT	Singolo paziente - Uso multiplo
LT	Vienas pacientas – daugkartinis naudojimas
NL	Eén patiënt - meervoudig gebruik
NO	Kun til bruk på én pasient – flergangsbruk
PL	Wielokrotne użycie u jednego pacjenta”
PT	Paciente Único - várias utilizações
PT-B	Paciente único - várias utilizações
RO	Pacient unic - Utilizare multiplă
SV	En patient – flera användningar
SK	Jeden pacient – viacnásobné použitie
SL	En bolnik - večkratna uporaba
TL	bawat pasyente - ng iba
TR	Tek Hasta - Çoklu Kullanım

UDI Unique Device Identifier

ES	Identificación única de producto
DE	Eindeutige Gerätekennung
DA	Unik enhedsidentifikator
FR	Identifiant de dispositif unique
IT	Identificatore univoco del dispositivo
LT	Unikalus įrenginio identifikatorius
NL	Unieke identificatiecode van het hulpmiddel
NO	Unik enhetsidentifikator
PL	Unikalny identyfikator urządzenia
PT	Identificador de dispositivo único
PT-B	Identificador de dispositivo único
RO	Identificator unic al dispozitivului
SV	Unik enhetsidentifierare
SK	Unikátny identifikátor pomôcky
SL	Edinstveni identifikator pripomočka
TL	Tulali banta identifier
TR	Benzersiz Cihaz Tanımlayıcı

Consult Instructions for Use

ES	Consultar instrucciones de uso
DE	Lesen Sie die Gebrauchsanweisung
DA	Se brugsanvisningen
FR	Consulter les instructions d'utilisation
IT	Consultare le istruzioni per l'uso
LT	Pasikonsultuokite naudoti naudojimą instrukcijas
NL	Raadpleeg de instructies voor gebruik
NO	Kontakt instruksjonene for bruk
PL	Skonsultuj się z instrukcjami do użycia
PT	Consulte as instruções de uso
PT-B	Consulte as instruções de uso
RO	Consultați instrucțiunile de utilizare
SV	Konsultera bruksanvisningen
SK	Konzultujte pokyny na použitie
SL	Oglejte si navodila za uporabo
TL	Sasangguni tagubilin gamitin
TR	Kullanım için talimatlara danışın

Latex-Free (Not Made with Natural Rubber Latex)

ES	Sin látex
DE	Latexfrei
DA	Latex fri
FR	Sans latex
IT	Senza latex
LT	Be latekso
NL	Latex vrij
NO	Uten latex
PL	Bez lateksu
PT	Sem Látex
PT-B	Sem látex
RO	Fără latex
SV	Latex fri
SK	Bez latexu
SL	Brez lateksa
TL	Walang Latex
TR	Lateks içermez

MD Medical Device

ES	Producto sanitario
DE	Medizinprodukt
DA	Medicinsk udstyr
FR	Dispositif médical
IT	Dispositivo medico
LT	Medicinos priemonė
NL	Medisch hulpmiddel
NO	Medisinsk utstyr
PL	Wyrób medyczny
PT	Dispositivo médico
PT-B	Dispositivo médico
RO	Dispozitiv medical
SV	Medicinteknisk produkt
SK	Zdravotnícka pomôcka
SL	Medicinski pripomoček
TL	Gamit Medikal
TR	Tıbbi cihaz

**Importer**

ES	Importador
DE	Importeur
DA	Importør
FR	Importateur
IT	Importatore
LT	Importuotojas
NL	Importeur
NO	Importør
PL	Importer
PT	Importador
PT-B	Importador
RO	Importator
SV	Importör
SK	Dovozca
SL	Uvoznik
TL	Mang-aangkat
TR	İthalatçı

**Keep Dry and Away from Direct Sunlight**

ES	Manténgase seco y alejado de la luz solar directa
DE	Trocken lagern und vor direkter Sonneneinstrahlung schützen
DA	Hold dig tør og væk fra direkte sollys
FR	Garder sèche et loin de la lumière directe du soleil
IT	Mantieniti asciutto e lontano dalla luce solare diretta
LT	Laikykite sausas ir toli nuo tiesioginių saulės spindulių
NL	Droog en uit de buurt van direct zonlicht houden
NO	Hold deg tørr og vekk fra direkte sollys
PL	Przechowywać w suchym miejscu i z dala od bezpośredniego światła słonecznego
PT	Mantenha seco e longe da luz solar direta
PT-B	Manter seco e afastado da luz direta do sol
RO	Păstrați uscare și departe de lumina directă a soarelui
SV	Håll torr och bort från direkt solljus
SK	Uchovávať v suchu a mimo dosahu priameho slnečného svetla
SL	Hranite suho in stran od neposredne sončne svetlobe
TL	At ang tuyong ang magtuturo sa sunlight
TR	Kuru ve doğrudan güneş ışığından uzak durun

LANGUAGES INDEX

ES	Spanish; ESPAÑOL
DE	German; DEUTSCH
DA	Danish; DANSK
FR	French; FRANÇAIS
IT	Italian; ITALIANO
LT	Lithuanian; LIETUVŲ KALBA
NL	Dutch
NO	Norwegian; NORSK
PL	Polish; POLSKI
PT	Portuguese; PORTUGUÊS (Portugal)
PT-B	Portuguese – Brazil; PORTUGUÊS – Brazil
RO	Romanian; ROMÂNĂ
SV	Swedish; SVENSKA
SK	Slovak; SLOVENSKÝ JAZYK (Slovenčina)
SL	Slovenian; SLOVENŠČINA
TL	Tagalog
TR	Turkish; TÜRKÇE

Do Not Use Bleach

ES	No utilice lejía
DE	Kein Bleichmittel verwenden
DA	Brug ikke blegemiddel
FR	N'utilisez pas d'eau de Javel
IT	Non utilizzare la candeggina
LT	Nenaudokite baliklių
NL	Gebruik geen bleekmiddel
NO	Ikke bruk blekemiddel
PL	Nie używać wybielaczy
PT	Não usar lixívia
PT-B	Não usar lixívia
RO	Nu folosiți înălbitor
SV	Använd inte blekmedel
SK	Nepoužívajte bielidlo
SL	Ne uporabljajte belila
TL	
TR	Çamaşır suyu kullanmayın

Caution or Warning

ES	Precaución o advertencia
DE	Vorsicht oder Warnung
DA	Forsigtighed eller advarsel
FR	Attention ou Avertissement
IT	Attenzione o avvertimento
LT	Įspėjimas arba įspėjimas
NL	Voorzichtig of waarschuwing
NO	Forsiktig eller advarsel
PL	Uwaga lub ostrzeżenie
PT	Cuidado ou Advertência
PT-B	Cuidado ou Advertência
RO	Atenție sau avertisment
SV	Försiktighet eller varning
SK	Upozornenie alebo výstraha
SL	Previdnost ali opozorilo
TL	
TR	Dikkat veya Uyarı

Date of Manufacture

ES	Fecha de fabricación
DE	Datum der Herstellung
DA	Fremstillingsdato
FR	Date de fabrication
IT	Data di produzione
LT	Pagaminimo data
NL	Productiedatum
NO	Produksjonsdato
PL	Data produkcji
PT	Data de fabrico
PT-B	Data de fabricação
RO	Data de fabricație
SV	Tillverkningsdatum
SK	Dátum výroby
SL	Datum izdelave
TL	
TR	Üretim tarihi